

Veterinary Instrumentation

Distington House, 26 Atlas Way, Sheffield U.K. S4 7QQ


t 0345 130 9596 Overseas t +44 114 258 8530

Email info@vetinst.com Website: www.vetinst.com



A covetrus Company

Trucut Supercore Type Biopsy Needles - Instructions for Use

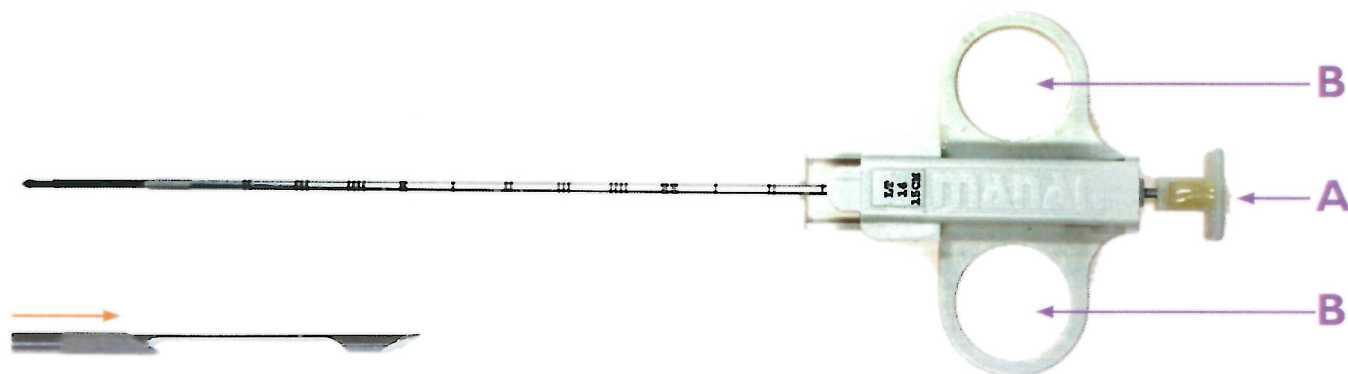

Images show Supercore Style

- Trucut Supercore Type biopsy needles are spring loaded. The cutting blade is forced, at speed, in the direction of the arrow (marked in orange) to cut and capture a sample of soft tissue.
- Before use the needle must be 'cocked'. Push thumb and finger through holes B and pull tab A until it clicks. The cutting blade now slides freely up and down the shaft.
- Once the area for biopsy is exposed the tab A is pushed gently up to holes B and the needle is held like a syringe just before injecting.
- The needle is introduced into a representative area and tab A is pushed against the handle including holes B again very much in the manner of a syringe. Firm pressure will result in the cutting part of the needle being fired down the shaft cutting and capturing the sample.
- Re-cocking the needle with expose and release the biopsy.
- Repeat as necessary to obtain a representative selection of biopsies.

Vi Product Codes

STB18 Vi Trucut Supercore Style Biopsy Needle 14g x 90mm

STB18B Vi Trucut Supercore Style Biopsy Needle 18g x 90mm

Trucut Biopsy Needles are also available non spring loaded - fully manual 14g x 115mm - Vi product code 012521